


智慧貓砂

每次如廁都是一次健康檢查 


MIAO MAIO TEST 尿潛血檢測顆粒

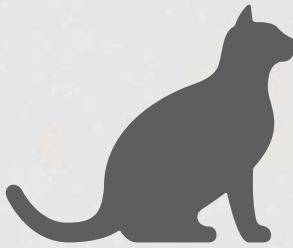

多達30種以上的泌尿道疾病都有尿(糞) 潛血的徵兆

成年貓最常見的疾病之一是貓泌尿道疾病，平均醫療費用超過5000台幣。 研究表明，在泌尿道感染的早期階段，即時檢測、介入治療和護理可以減少疾病發展引起的身體疼痛和治療成本。

貓咪各種泌尿系統疾病的早期預警之一是尿潛血(血尿)(Hematuria)。 貓咪的血尿可以透過MIAO MIAO TEST 尿潛血檢測顆粒準確檢測，該測試即時向其飼主提供預警。

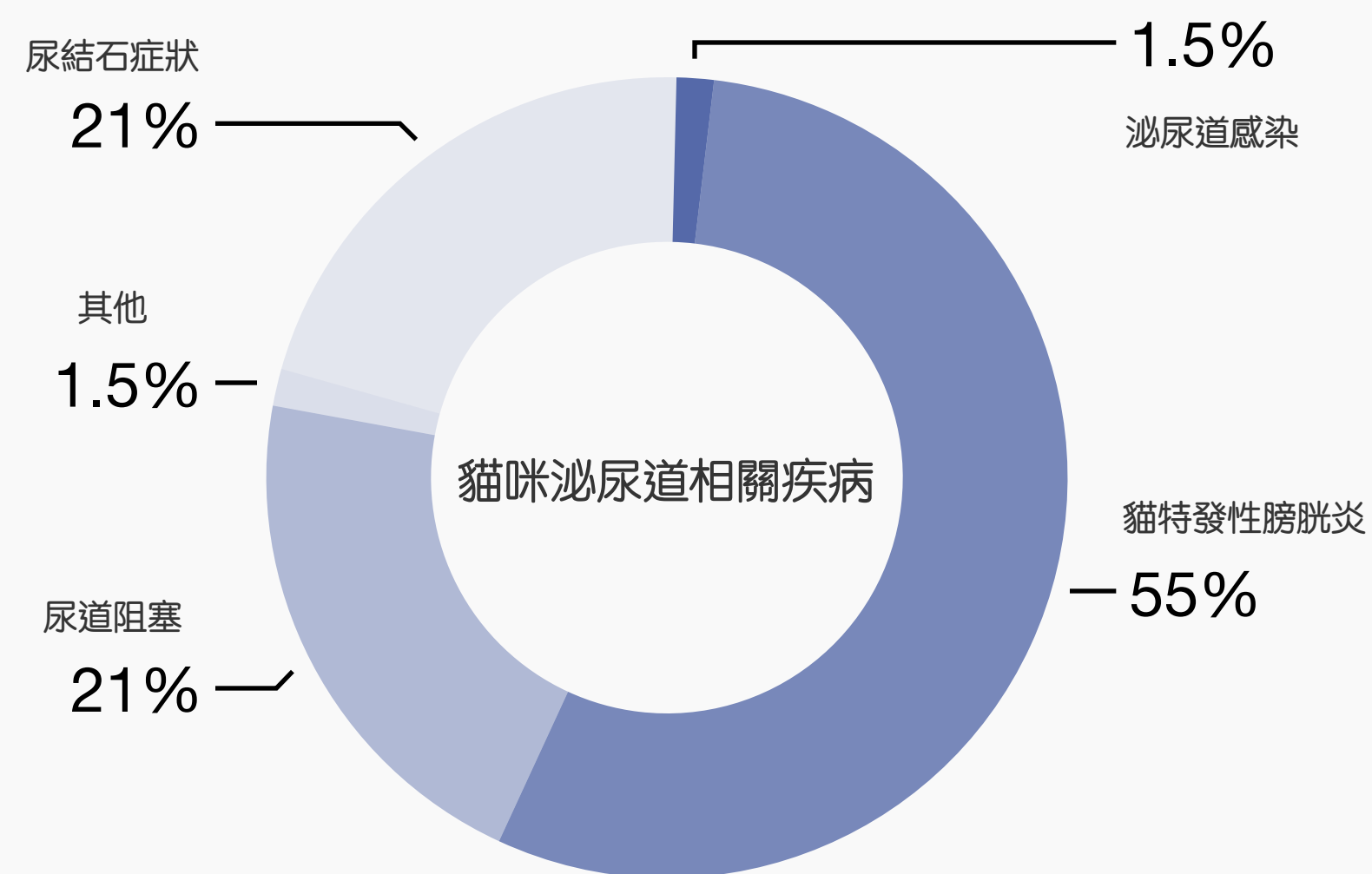

尿(糞)潛血
(肉眼不可見)

MIAO MIAO TEST 尿潛血檢測顆粒

精準檢測貓咪尿潛血


核心技術 - "危險因子 (尿糞便潛血)" 尿液中微量血液藉由MIAO MIAO TEST 尿潛血檢測顆粒被檢出。貓砂加入尿潛血檢測顆粒的檢測功能提供飼主在家自我日常了解貓咪泌尿道健康。

藍色顯示陽性結果

檢測顆粒與尿液中血紅蛋白反應，顆粒顏色立即由白轉成藍。

清晰顯色

專利隔氧三層結構；顯色清晰，飼主容易判讀結果。

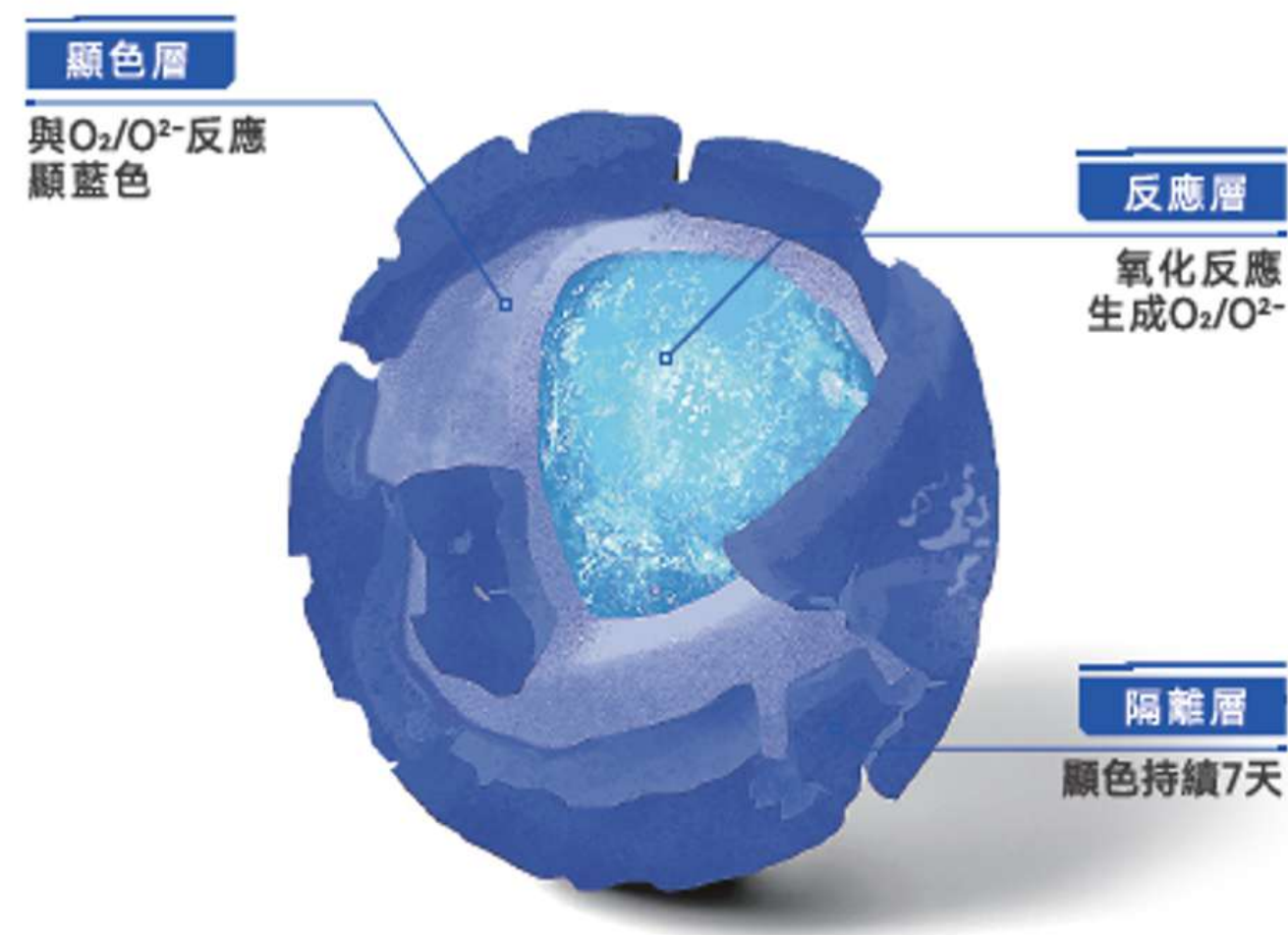

顆粒結構



顯色靈敏 測試結果容易判讀

- 檢測顆粒對尿液中血含量的反應不同，觀察顆粒顯示的顏色強度可以簡單評估貓咪血尿嚴重程度。
- 對於不同程度的尿潛血，可以採取相應的改善措施，如及時用就醫、調整飲食、促進水的攝入、改善環境狀況和避免壓力。



1/100
含血量

非常危險
立即就醫

1/1000
含血量

危險
儘速安排就醫

1/10000
含血量

血尿示警
血尿症狀明顯，預約就醫

1/100000
含血量

日常改善
與專業醫師討論改善計畫

3秒顯色

顏色顯示可持續長達7天

- 檢測顆粒反應時間只需3秒。(血液含量的提高，顏色變化更快更深)。
- 獨特的三層結構——反應層、隔離層和顏色顯示層。隔離空氣，將顏色顯示時間延長到7天左右。確保了從工作或旅行回來的寵物主人可以即時檢視測試結果。



靈敏的顏色反應

十萬分之一的含血量就能被檢測出

根據第三方公正組織的實驗結果（ eurofins），MIAO MIAO TEST 尿潛血檢測顆粒的靈敏性得到肯定。
秒秒測檢測顆粒可以在1/100,000的尿中血液濃度下檢測出含血反應。





URINARY OCCULT BLOOD TEST
GRANULES RESULT

Client Name: Lianshui Jinnannong Biotechnology Co., Ltd
Sample Code: 960-2022-00007809, Miaomiaoce Urinary Occult Blood Test Granules
Reference Method: Internal control standards of Urinary Occult Blood Test Granules
Analysis Date: 01-Dec-2022 to 22-Dec-2022
Report Date: 23-Dec-2022

Summary of Test Result(s)

Item	Reagent	Result	Conclusion
Specificity	DI Water	No discoloration	Satisfactory
	DI Water	No discoloration after 7 days	Satisfactory
	One ten-thousandth of cat blood diluent	Turned light blue within 3 seconds, and dark blue after 30 minutes.	Satisfactory
	One ten-thousandth of cat blood diluent	No color fading after 7 days	Satisfactory
Sensitivity	DI Water	No discoloration	Satisfactory
	DI Water	No discoloration after 24 hours	Satisfactory
	DI Water	No discoloration after 15 days	Satisfactory
	One ten-thousandth of cat blood diluent	Dark blue	Satisfactory
	One ten-thousandth of cat blood diluent	No color fading after 24 hours	Satisfactory
	One ten-thousandth of cat blood diluent	No color fading after 15 days	Satisfactory
	One hundred-thousandth of cat blood diluent	Light blue and white	Satisfactory
	One hundred-thousandth of cat blood diluent	No color fading after 24 hours	Satisfactory
	One hundred-thousandth of cat blood diluent	No color fading after 15 days	Satisfactory

The above items inspection conclusion: Pass

Analysts: 张以宁 Approver: 罗仁宏

END OF REPORT

Eurofins Testing Inspection Certification (Chendu) Co.,Ltd
F8/9, Block B2,
No.18 Bio-town, Middle Road Section Two,
Shuangliu District, Chengdu, Sichuan, China
Postal code: 610219

Phone: +86 28 81190137
www.eurofins.com





智慧貓砂

貓砂進入「檢測時代」


從那時起，貓砂不僅是一種清潔產品，而且還具有尿液測試的功能。

它標誌著寵物預防醫學的新時代——“檢測時代”！


我們的目標是不斷為貓咪保健提供負擔得起、有效和方便的工具。

團隊的願景，為貓的健康做出貢獻，並將預防醫學的概念帶給更多養寵物的家庭！


Dr.vet[®] 秒秒測 檢測顆粒

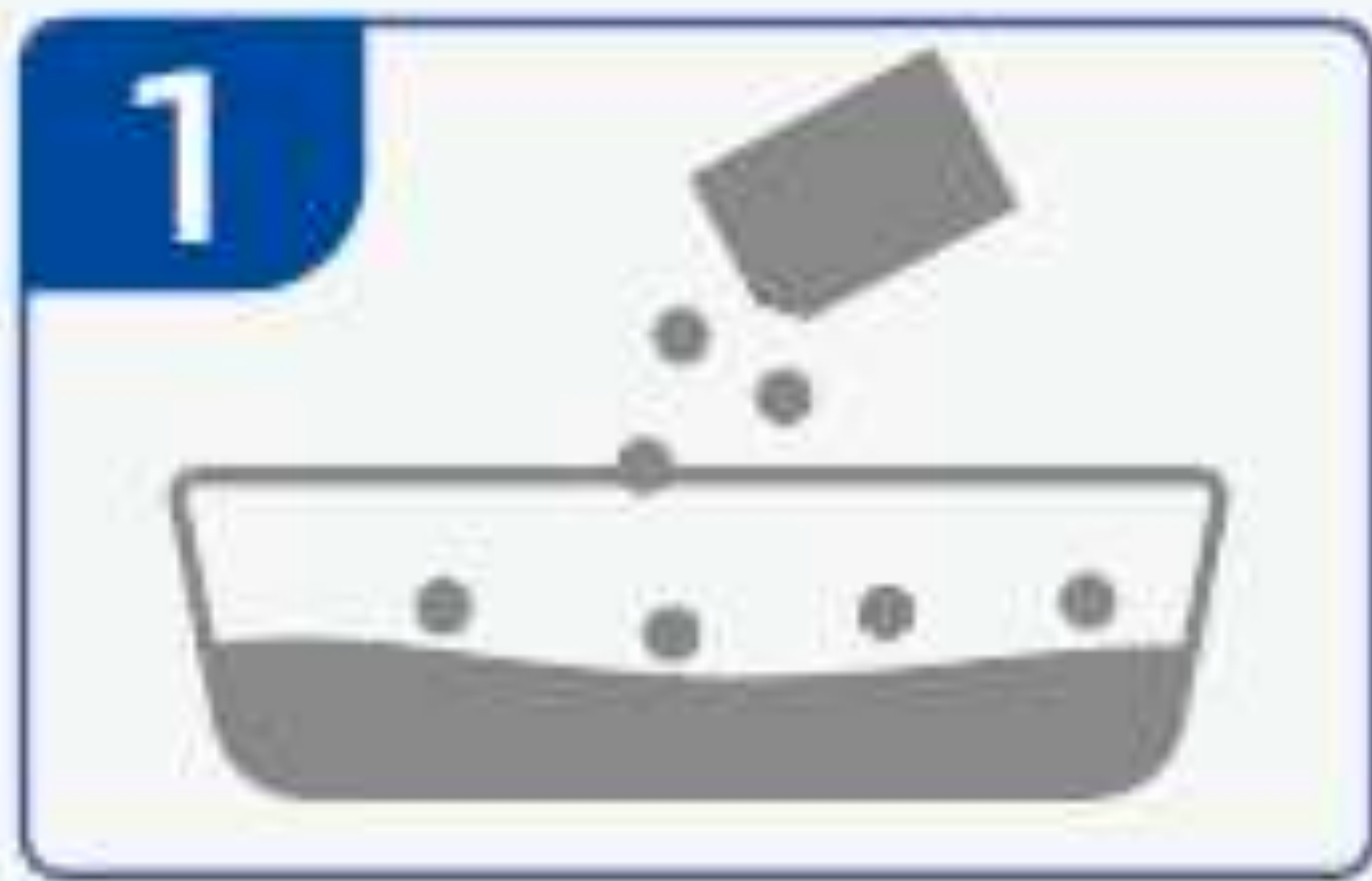

檢測顆粒平均灑在貓砂表面

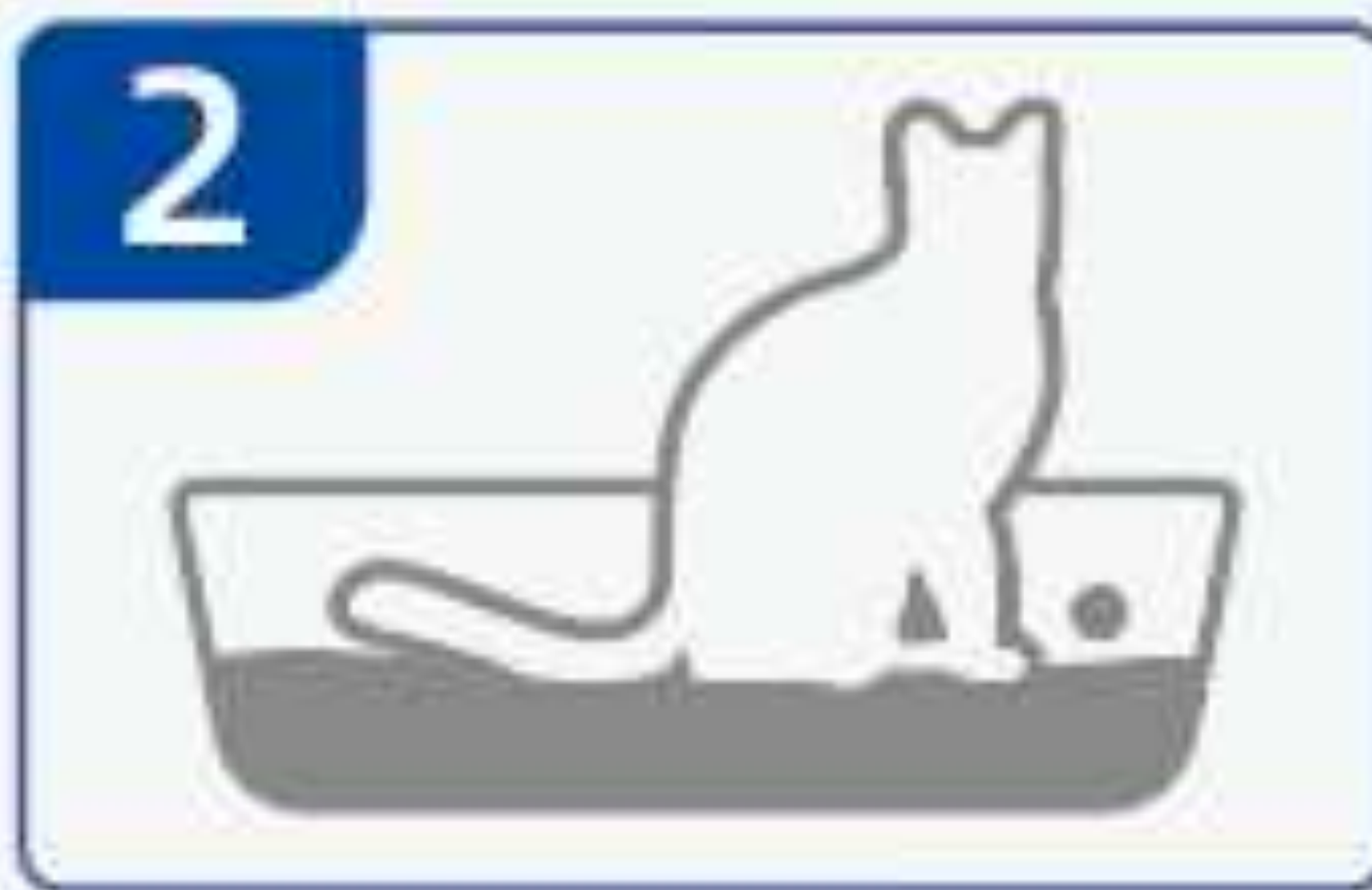

貓咪尿尿後，檢查貓砂盆

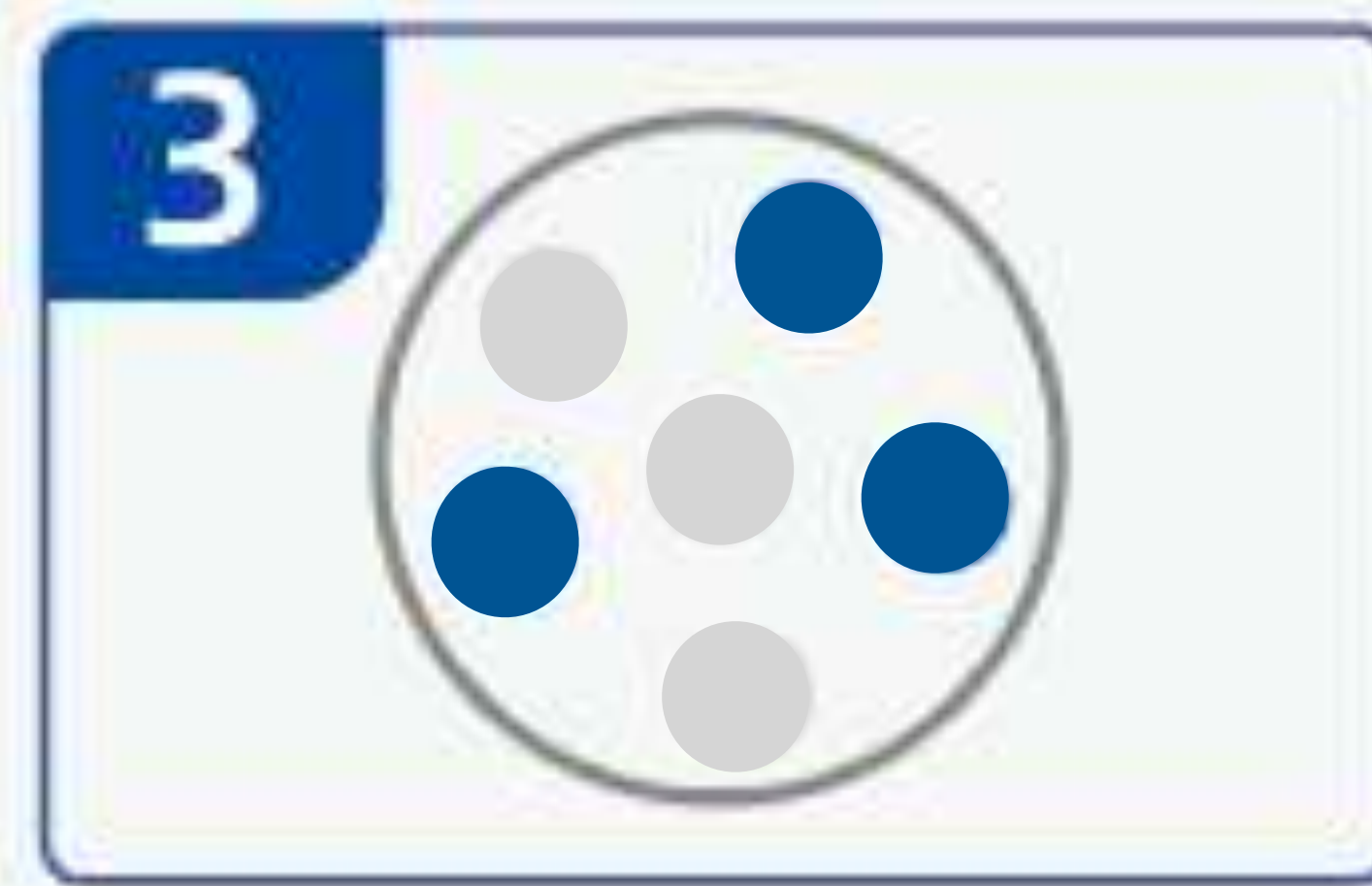

顆粒顯示藍色，洽詢您的獸醫師

Check!



產品規格

- 產品名稱：MIAO MIAO TEST 尿潛血檢測顆粒
- 包裝量：10克 (每袋)
- 保存期限：24個月
- 保存條件：陰涼乾燥環境，避免陽光照射
- 包裝組成：10克/袋, 10袋/盒


本品與純豆腐砂、植物砂與礦砂適配度最好，不建議與其他類型貓砂混用。
請避免接觸強酸、強鹼、強氧化物(如除臭顆粒、除臭劑、消毒劑等)。

本產品不具任何疾病診斷及治療功能，若顏色異常請諮詢醫生。





Jinhoo Technology CO. LTD.
www.jinhootech.com.tw
service@jinhootech.com.tw

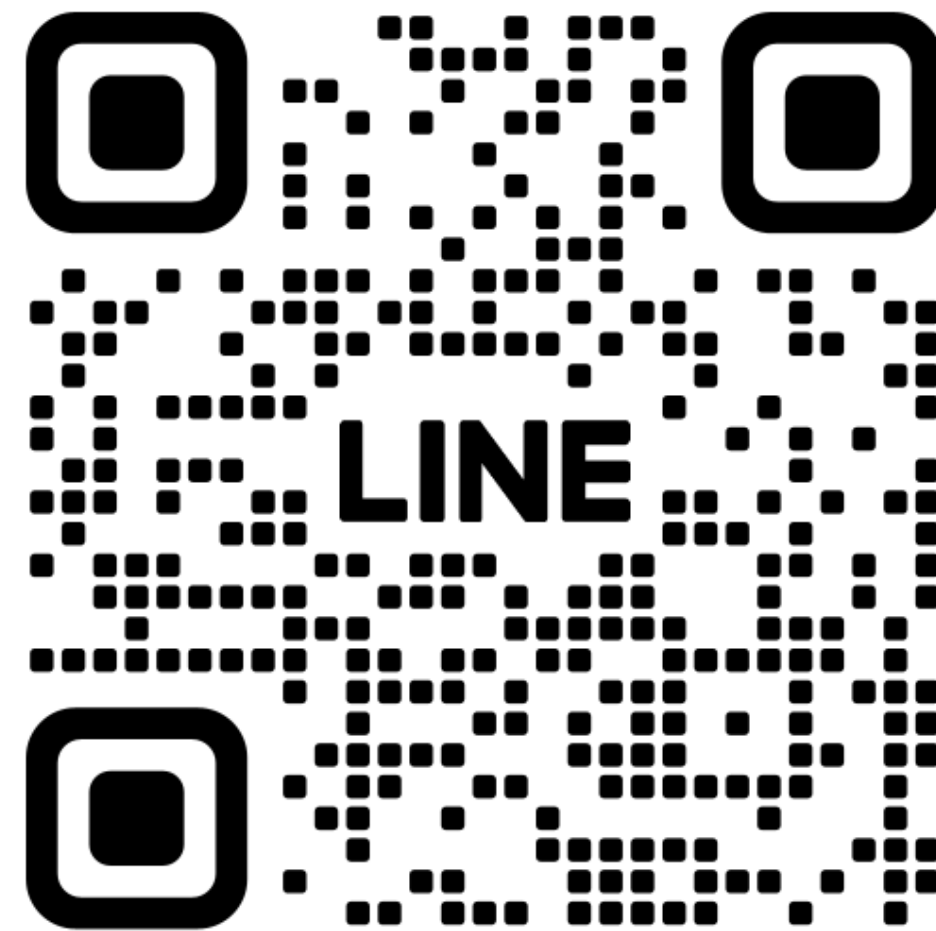

Dr.vet[®]

Provide a urinalysis every time your cat uses the litter box.


Hematuria (Urine occult blood) test series



(Occult) Blood in your cat's urine Indicate 30+ urinary tract diseases

One of the most common diseases of adult cats is feline urinary tract disease, and the average medical costs more than 170USD. Research shows that timely detection, interventional treatment and nursing in the early stage of urinary tract infection can reduce the physical pain and treatment cost which caused by the development of the disease.

One of the early warning of various urinary diseases in cats is urine occult blood. Hematuria in cats can be accurately detected by Dr.Vet urine occult Miao Miao test which serves an early warning to their owners in time.

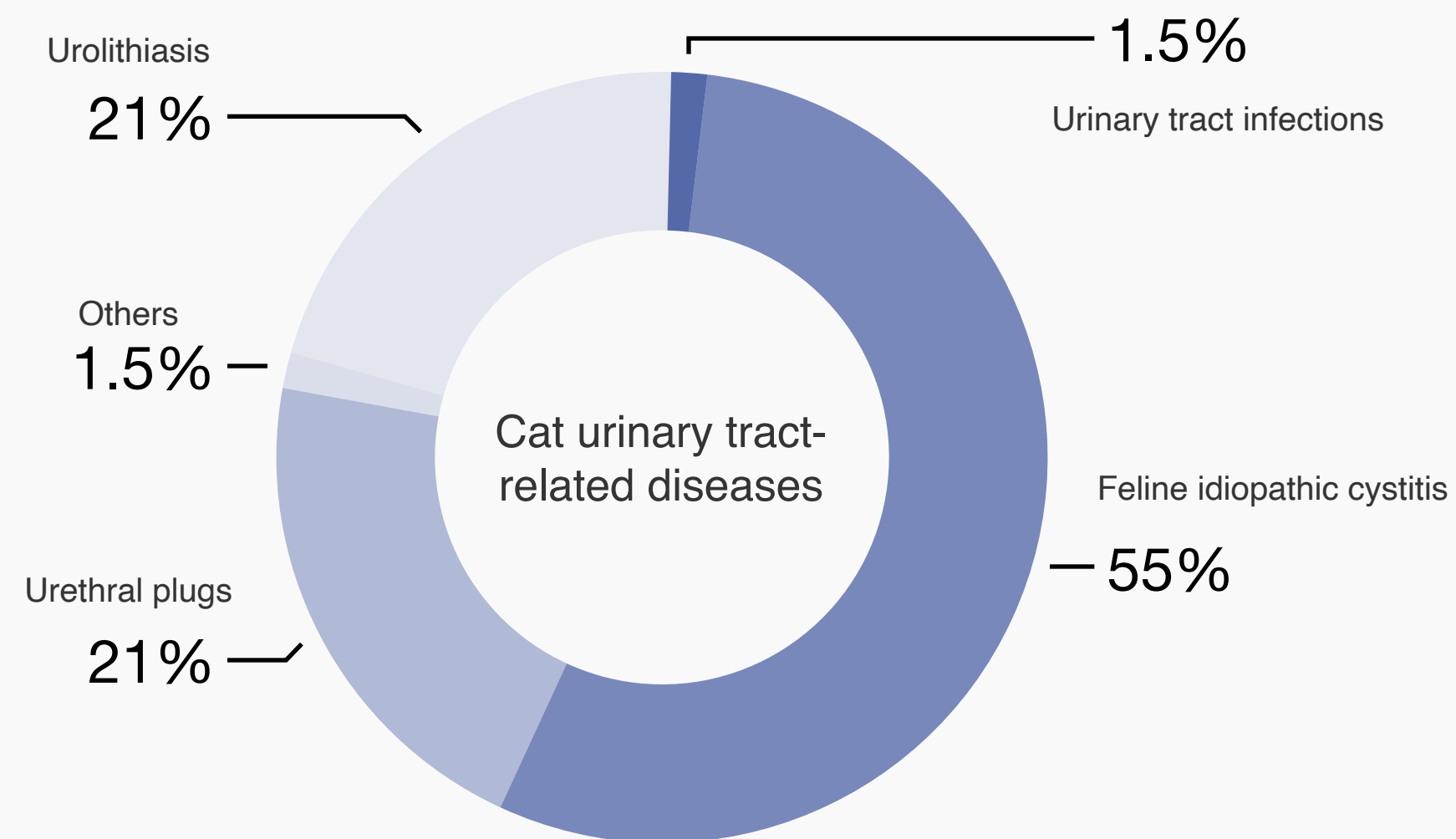

urine occult blood
(Invisible to the naked eye)

Miao Miao Test

Detect feline urine occult blood accurately

The core technology - "Dangerous factor (urine occult blood)" can be detected sensitively by Miao Miao Test particles in cat urine. The cat litter with hematuria detection particles provides a daily self-inspection for feline urinary health at home.

Blue shows positive results

Detecting particles react with hemoglobin and the color shifts from white to blue.

Clear results

The unique three-layer structure with an oxygen barrier layer enhances color vibrancy and makes it easier to distinguish.

COLOUR LAYER


react with O_2/O^{2-} and turn blue

REACTION LAYER

oxidation reaction
Produce O_2/O^{2-}

PROTECTION LAYER

Colour remains over 7 days

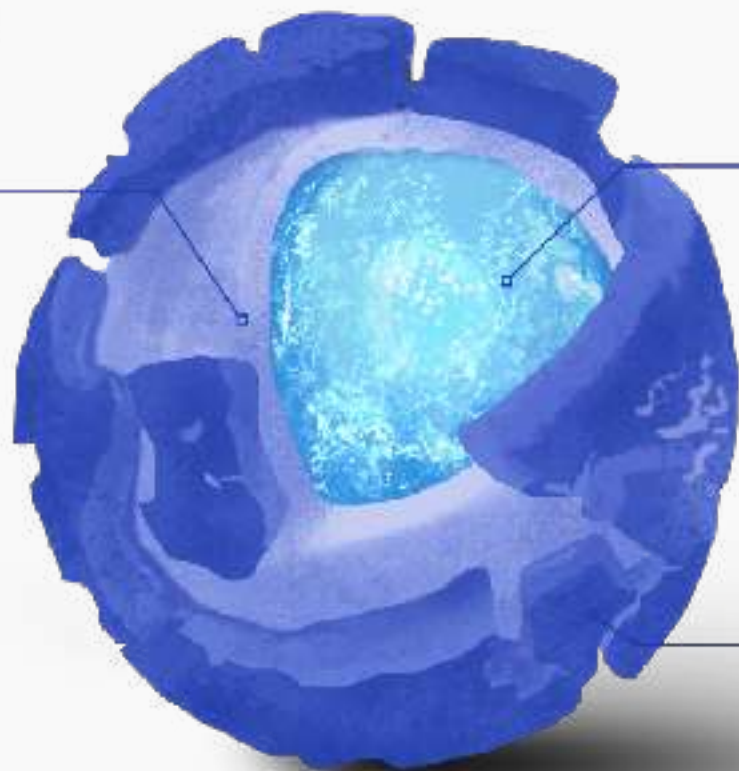

The structure of Miao Miao Test particles



Sensitive Color display

Readable test result with ease

- Detection of particles reacts differently to urinary blood content, and Observing the color intensity of the displayed particles allows for a simple assessment of the severity of hematuria in cats.
- For different levels of hematuria, corresponding improvement measures can be taken, such as timely medication, dietary adjustments, promoting water intake, improving the environmental condition, and avoiding stress.



1/100
Blood content

Very dangerous
Medical emergency
scenario

1/1000
Blood content

Dangerous
Medical attention ASAP

1/10000
Blood content

Health warning
Obviously hematuria

1/100000
Blood content

**Dietary
management**
Increasing water intake
Reducing stress
Consult the medical
professional

3-Second color display ⚡ Clear results lasting for 7 days

- Particle reaction time for detection is only 3 seconds. (Faster color change with higher blood content).
- The unique three-layer structure — a reaction layer, isolation layer, and color-display layer, isolating air, extending the color display time to around 7 days. This ensures that pet owners returning from work or trips can view the test results in time.



Accurate color reaction

Sensitivity of 1/100,000 dilutions of blood

According to the experimental result by third-party impartial organization ( eurofins), the sensitivity of Miao Miao Test urine occult blood detection particles was proved. 1/100,000 dilution of blood can be indicated by Miao Miao Test particles.





URINARY OCCULT BLOOD TEST
GRANULES RESULT

Client Name: Lianshui Jinnannong Biotechnology Co., Ltd
Sample Code: 960-2022-00007809, Miaomiaoce Urinary Occult Blood Test Granules
Reference Method: Internal control standards of Urinary Occult Blood Test Granules
Analysis Date: 01-Dec-2022 to 22-Dec-2022
Report Date: 23-Dec-2022

Summary of Test Result(s)

Item	Reagent	Result	Conclusion
Specificity	DI Water	No discoloration	Satisfactory
	DI Water	No discoloration after 7 days	Satisfactory
	One ten-thousandth of cat blood diluent	Turned light blue within 3 seconds, and dark blue after 30 minutes.	Satisfactory
	One ten-thousandth of cat blood diluent	No color fading after 7 days	Satisfactory
Sensitivity	DI Water	No discoloration	Satisfactory
	DI Water	No discoloration after 24 hours	Satisfactory
	DI Water	No discoloration after 15 days	Satisfactory
	One ten-thousandth of cat blood diluent	Dark blue	Satisfactory
	One ten-thousandth of cat blood diluent	No color fading after 24 hours	Satisfactory
	One ten-thousandth of cat blood diluent	No color fading after 15 days	Satisfactory
	One hundred-thousandth of cat blood diluent	Light blue and white	Satisfactory
	One hundred-thousandth of cat blood diluent	No color fading after 24 hours	Satisfactory
	One hundred-thousandth of cat blood diluent	No color fading after 15 days	Satisfactory

The above items inspection conclusion: Pass


Analysts: 张以宁 Approver: 罗仁宏

END OF REPORT

Eurofins Testing Inspection Certification (Chendu) Co.,Ltd
F8/9, Block B2,
No.18 Bio-town, Middle Road Section Two,
Shuangliu District, Chengdu, Sichuan, China
Postal code: 610219

Phone: +86 28 81190137
www.eurofins.com

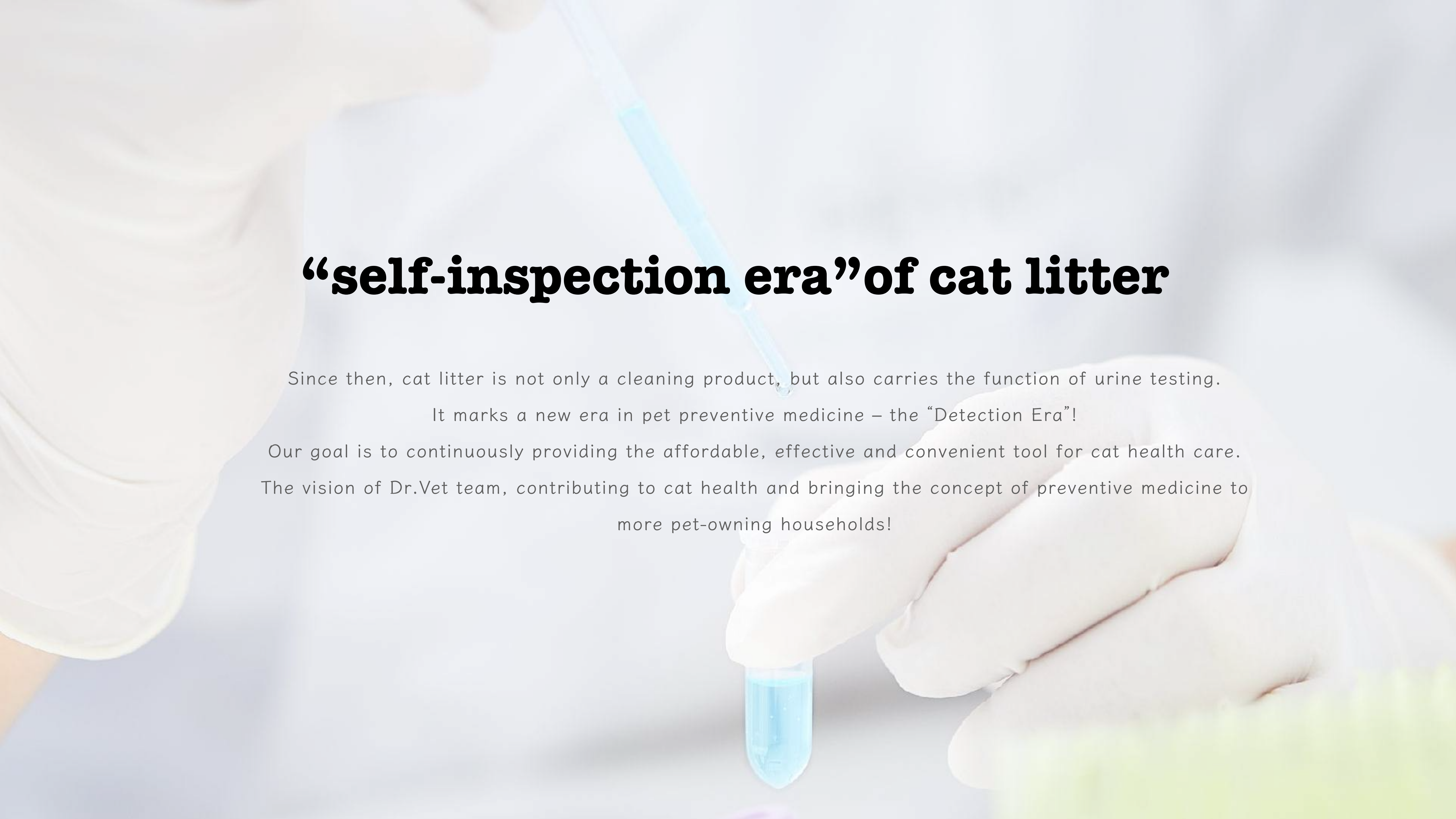

A close-up photograph of a person's hand wearing a white nitrile glove. The hand is holding a blue pipette, which is dispensing a small amount of blue liquid into a blue test tube. The test tube is also held by the gloved hand. The background is a soft, out-of-focus light blue and white. The text "self-inspection era" of cat litter is overlaid on the image.


“self-inspection era” of cat litter

Since then, cat litter is not only a cleaning product, but also carries the function of urine testing.


It marks a new era in pet preventive medicine – the “Detection Era”!

Our goal is to continuously providing the affordable, effective and convenient tool for cat health care.
The vision of Dr.Vet team, contributing to cat health and bringing the concept of preventive medicine to
more pet-owning households!


HOW TO USE

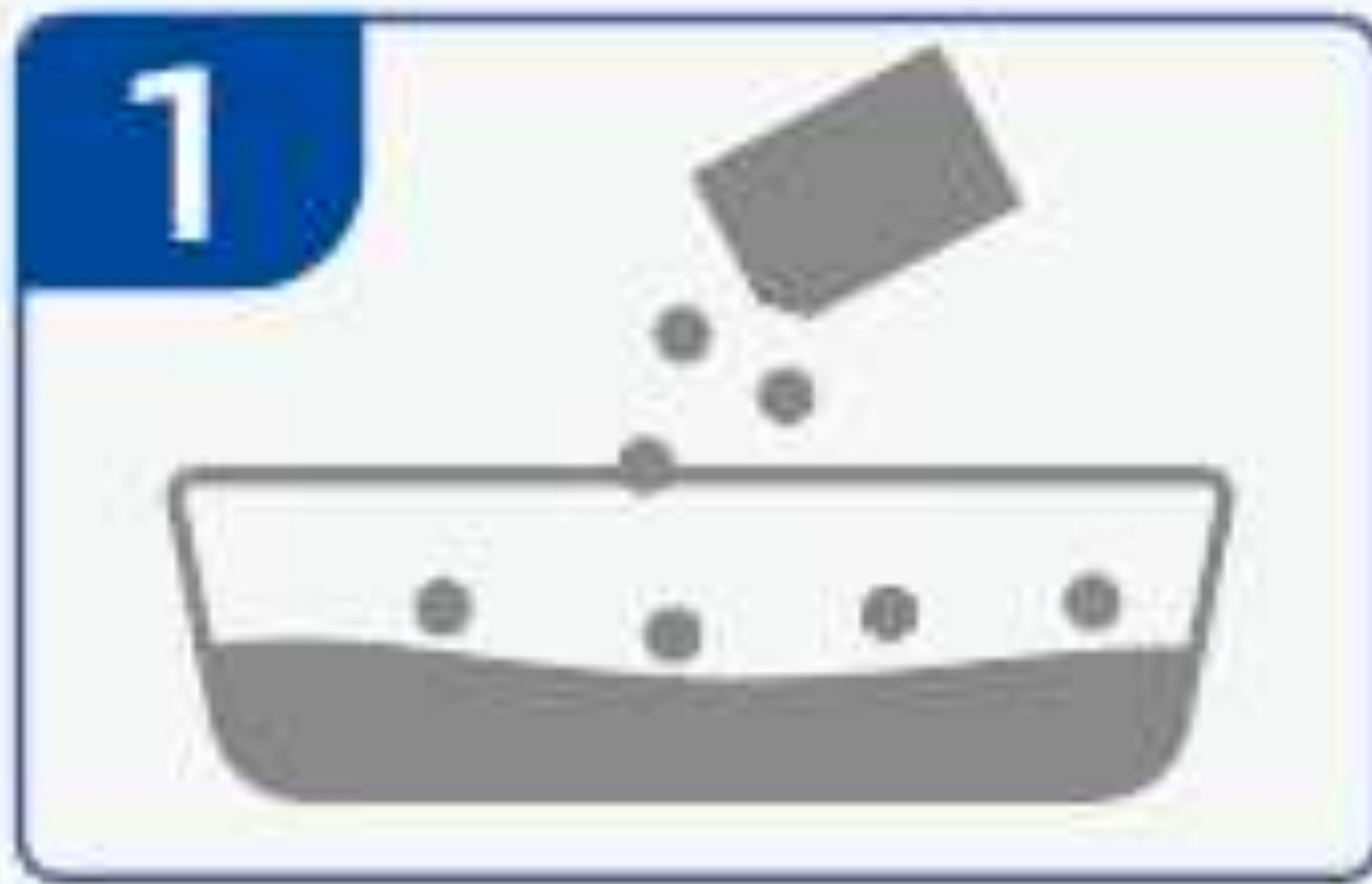


Spread the particle evenly on the cat litter surface

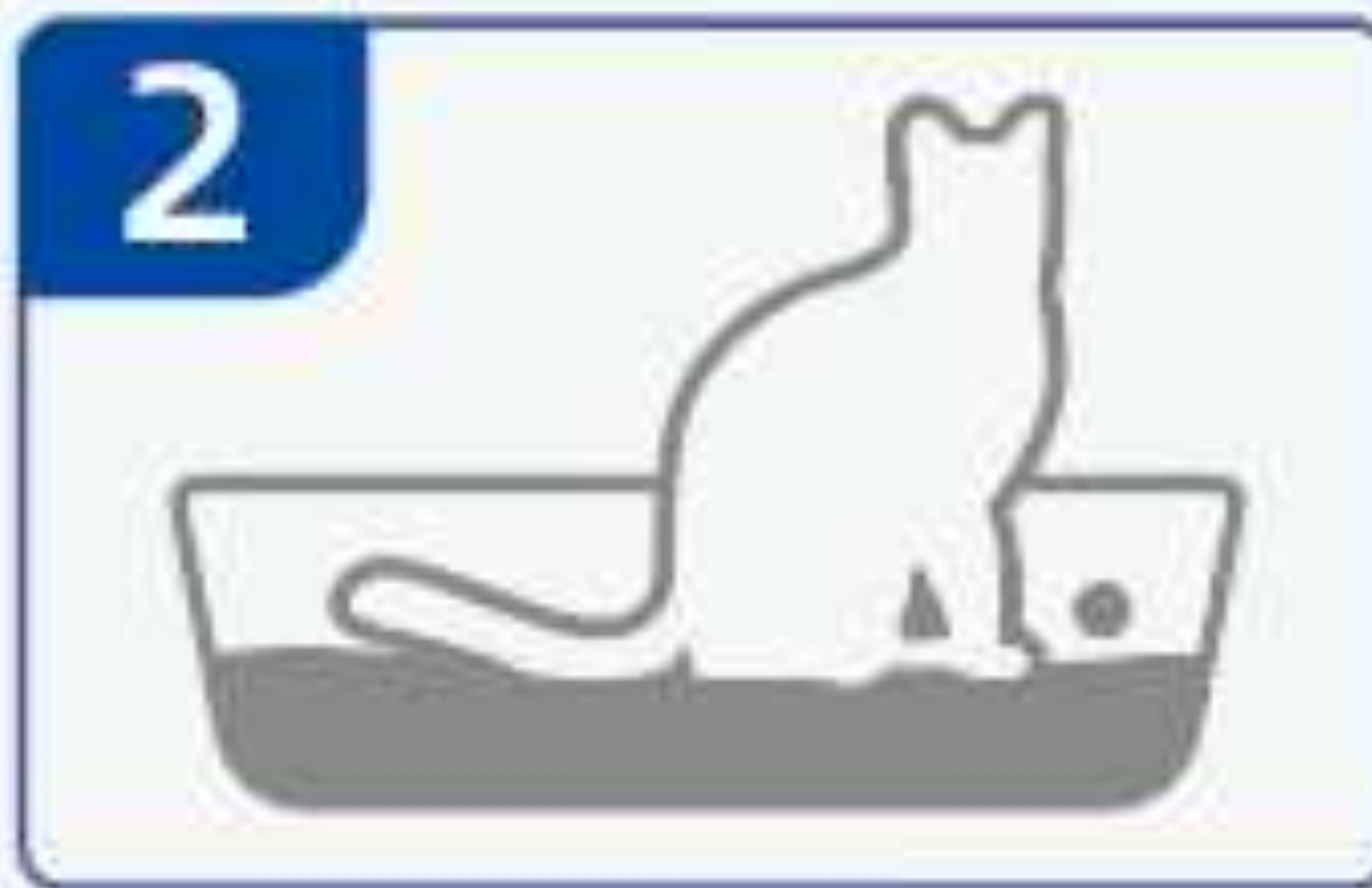

After cat urination, check the litter box

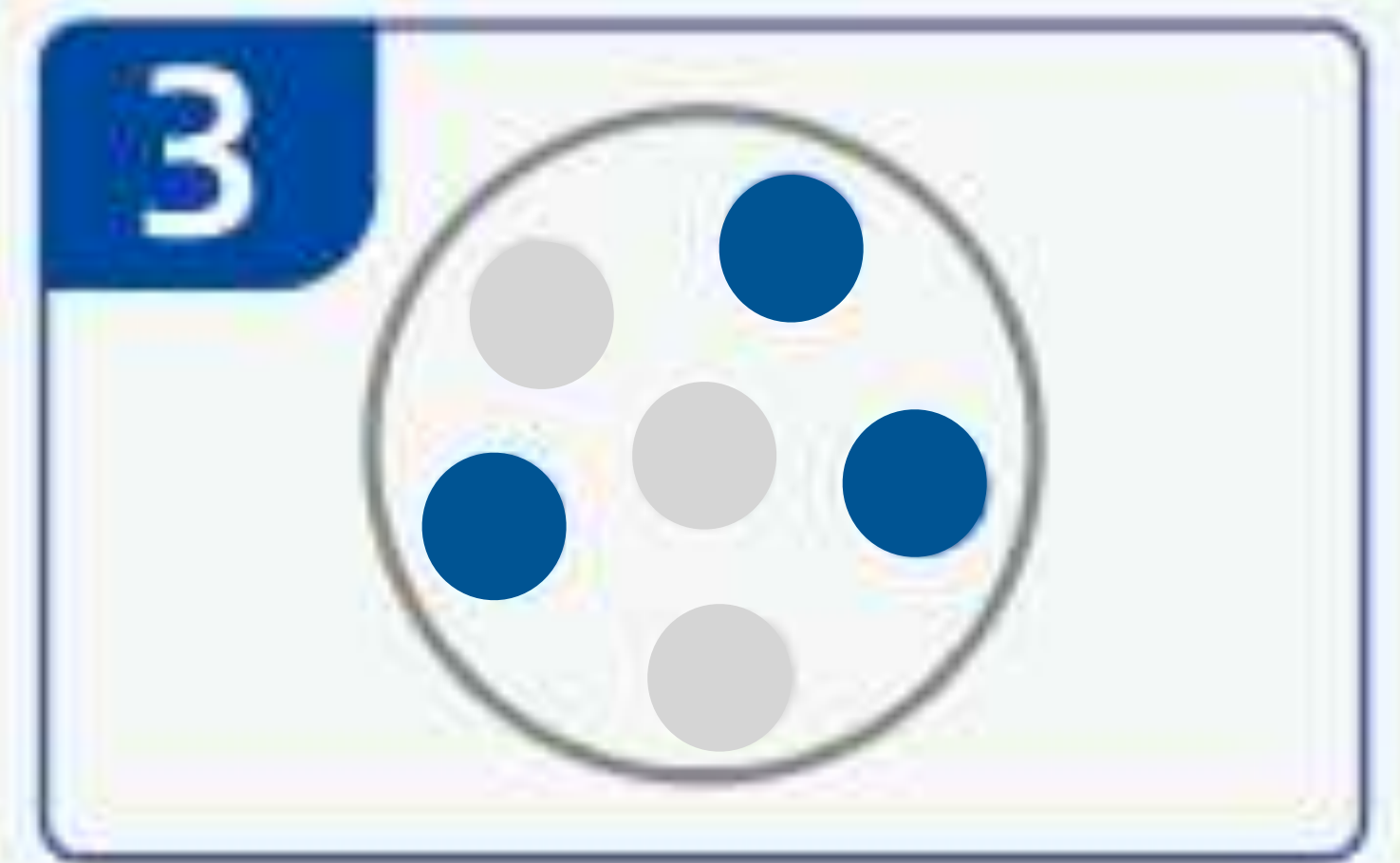

Consult a veterinarian if The color blue shows

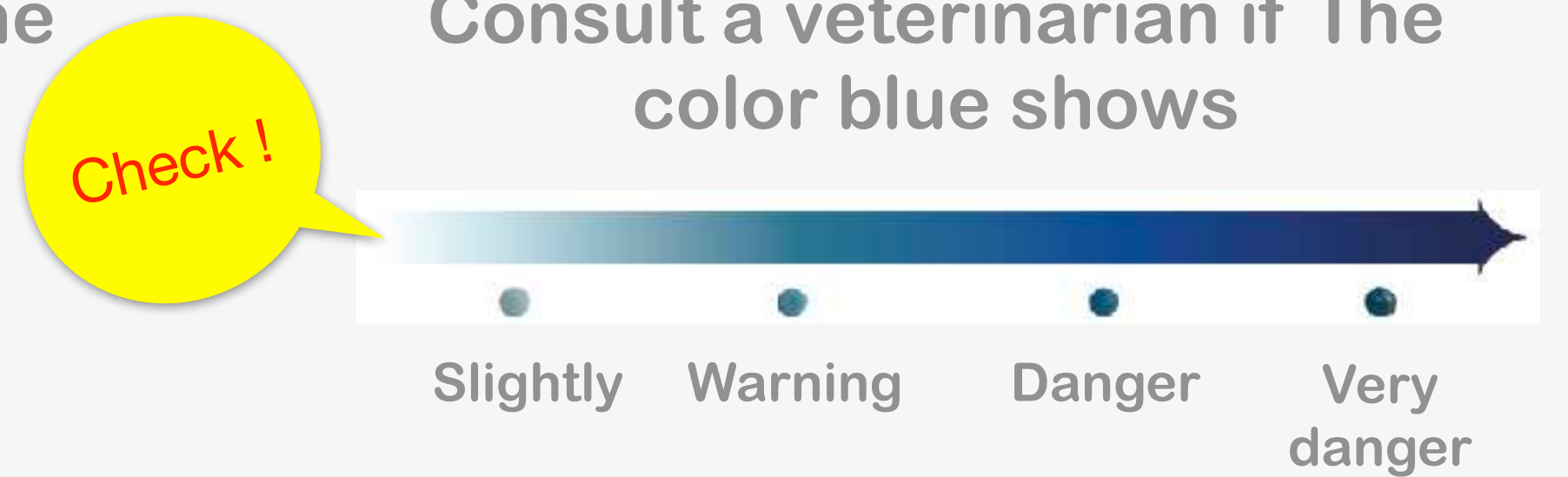

Product Specification

Product name : MIAO MIAO TEST
URINE OCCULT BLOOD TEST PARTICLES

Net weight : 10g (per bag)

Shelf life : 24 Months

Storage condition : Cool, dry place and avoid sun light

Packing : 10g/bag, 10bags/box





Jinhoo Technology CO. LTD.
www.jinhootech.com.tw
service@jinhootech.com.tw

